

Bed Sensor Mat

REDUCE THE RISK OF FALLS & INJURIES

LOW PROFILE, NON-SLIP MATERIAL

PORTABLE

EASY TO USE

EASY TO MOVE FROM ROOM TO ROOM

SOLVENT WIPE-DISINFECTANT


INTEGRATES TO MOST NURSE CALL SYSTEMS

PLUG & PLAY OPERATION

DURABLE & RELIABLE

COST EFFECTIVE

MEDIUM SIZED PRESSURE MAT (760mm X 510mm)

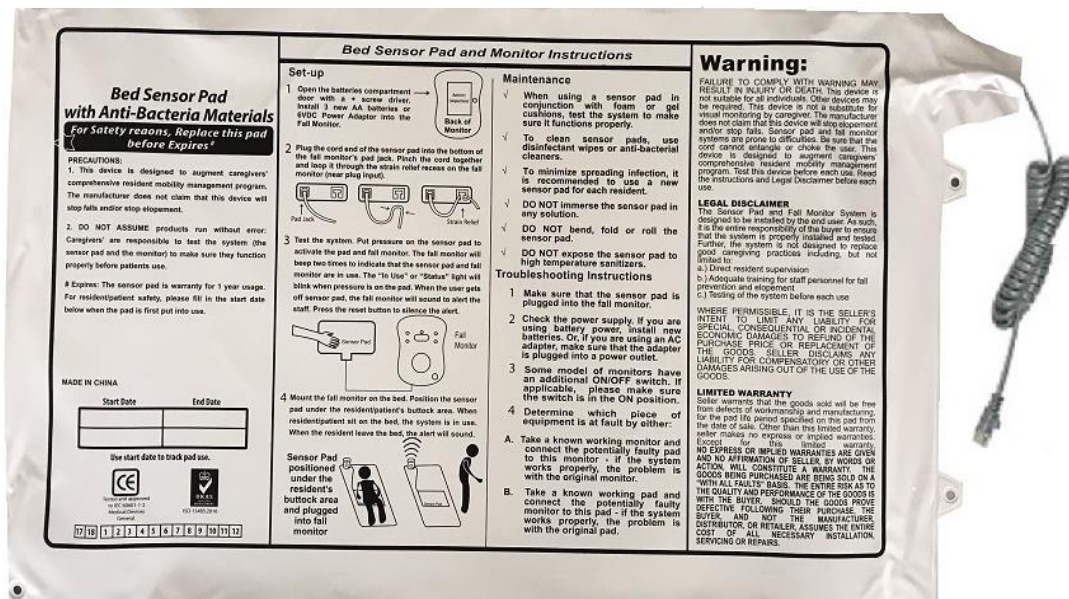

MED-NC-B365-NC20 (with RJ11)

MED-NC-B365-JS63 (with Jack 6.35mm)

(TGA Registration: 223902)

Indigo Care provides a pressure sensitive Premium Bed Sensor Mat that is designed to immediately alert staff should a resident leave their bed. This is most useful to residents who are at risk of falling out of their bed, or those who require assistance in arising from the bed. The mat is placed on the bed, directly under the bed linen. For a normally open call system, a monitor is connected to the bed sensor mat and then the mat is connected to your existing nurse call system with a Y Cable (via the monitor). Once the resident removes themselves from the mat and tries to leave the bed, the sensor mat will activate the monitor's alarm and alert the staff. Set-up instructions are printed on the actual mat. Eyelets at the end of the mat are for easy storage.

KEY SPECIFICATIONS:

- **Dimensions: 760mm X 510mm**
- **Connection: RJ11 on 500mm curly cord (normally open) or jack 6.35mm plug (normally close)**
- **Electrical resistance < 500 Ohm**
- **White PVC, Thickness 0.20mm**
- **Warranty: 12 months**

PRODUCT ACCESSORIES:

- **RJ11 requires Y Cable**
- **Jack 6.3 requires straight 6.35mm or 3.5mm cable**
- **Anti-Tamper Monitor (use 4 X AA batteries-not included)**
- **6V DC Adaptor (optional)**

Horizon Electronic Components T/as INDIGO CARE
2/71 Rusdale Street, Knoxfield, VIC, 3180
PH +61 3 9763-2988 FAX +61 3 9763-3811
www.indigocare.com.au
ABN 36007302801



ORDER INFORMATION

Fax or email your purchase orders to
03-9763-3811
Or email sales@indigocare.com.au

indigo

care